DERMATOPATHOLOGY GROSSING GUIDELINES

QI Project Potocol:

- Process margins separately as RUSH
 - For larger excisions which require consulting with attending still
 RUSH en face sampled margins or any margin sampled
- Use a case flag for cases that a HN surgeon performs the skin excision

RUSH CASES:

For all RUSH cases, the dermpath fellows AND the skin service resident should be notified via email about the case.

Specimen Type: DISCOID (circular) SKIN EXCISION (or other unusual shape other than ellipse)


Procedure:

Please contact the assigned pathologist prior to grossing in these specimens.

- 1. Measure the specimen in 3 dimensions.
- 2. If oriented, ink the specimen differentially using four colors.
 - a. The four colors will meet on the deep aspect of the specimen (see diagram, below). It is important that these colors extend to the deep aspect of the tissue, so that one can maintain orientation on the slide after the peripheral margins have been removed.



- 3. If unoriented, ink all peripheral and deep margins green.
- 4. Shave off the peripheral margins (as a 2 mm 'strip') by quadrant, and place in 4 separate cassettes.

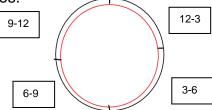

- 5. Serially section the remainder of the specimen and examine for depth of invasion. Up to 3 central sections can be submitted in each remaining cassette. If the specimen is large (will require greater than 20 cassettes to entirely submit), please consult with the Attending on service prior to processing the sample.
- 6. Enter embedding instructions in Task Notes in Beaker (embed inked side down and add deep section "H&E Deepcut Not REcut (150 micron waste)).—

 for the peripheral margins only.

DERMATOPATHOLOGY GROSSING GUIDELINES

Gross Template:

MMODAL COMMAND: INSERT DISCOID SKIN

It consists of an oriented circular skin excision measuring [***] cm (12:00-6:00) x [***] cm (3:00-9:00), and excised to a depth of [***] cm. There is a short suture designating [***] and a long suture designating [***]. The epidermis is remarkable for a [describe lesion/unremarkable***]. The lesion is located [***] cm from 12:00, [***] cm from 3:00, [***] cm from 6:00, and [***] cm from 9:00.

The peripheral margins are shaved and entirely submitted. The remaining specimen is serially sectioned to reveal a [describe cut surface/yellow-tan/fibrotic***] cut surface. The lesion is located [***] cm from the deep margin. The specimen is entirely submitted.

INK KEY:

Blue 12-3 o'clock peripheral and deep margins Red 3-6 o'clock peripheral and deep margins Green 6-9 o'clock peripheral and deep margins Orange 9-12 o'clock peripheral and deep margins

CASSETTE SUMMARY:

[***]1 12-3:00 periphery, shave [add H&E/Deepcut Not Recut (150 micron waste) section in Casebuilder***]

[***]2 3-6:00 periphery, shave [add H&E/Deepcut Not Recut (150 micron waste) section in Casebuilder***]

[***]3 6-9:00 periphery, shave [add H&E/Deepcut Not Recut (150 micron waste) section in Casebuilder***]

[***]4 9-12:00 periphery, shave [add H&E/Deepcut Not Recut (150 micron waste) section in Casebuilder***]

[***]5 Central sections

Cassette Submission: All tissue submitted

- If oriented- shave off peripheral margins (as a 2 mm 'strip') by quadrant and place in 4 separate cassettes. The remaining serially section central sections can be submitted entirely. Up to 3 central sections can be submitted in each remaining cassette.
- If unoriented- submit shaves of peripheral margins sequentially. The remaining serially sectioned central sections can be submitted entirely, sequentially. Up to 3 central sections can be submitted in each remaining cassette.

Sample Cassette Submission:

Oriented

A1 12 - 3:00 peripheral margin, shave, en face A2 3 - 6:00 peripheral margin, shave, en face A3 6 - 9:00 peripheral margin, shave, en face A4 9 - 12:00 peripheral margin, shave, en face

A5-A6 Central sections, submitted sequentially from 12 to 6 o'clock,

perpendicular sections

Unoriented

DERMATOPATHOLOGY GROSSING GUIDELINES

A1-A4	peripheral margins, shave, sequentially submitted, en face
A5-A6	Central sections, sequentially submitted perpendicular sections